ANIOSPRAY QUICK

ANIOSPRAY QUICK

Sommaire

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Regulation and general information

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***ANIOSPRAY QUICK** is designed, produced and controlled by the Laboratoires ANIOS, certified by the AFAQ Organism under the number 1995/3723 in accordance with the ISO 9001 Quality System*

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***ANIOSPRAY QUICK** is in compliance with the requirements of the European regulation related to biocides for disinfection of surfaces (Group 1 - PT 2)*

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***ANIOSPRAY QUICK** has the CE mark for medical devices in the category class IIa, in accordance with the Directive 93/42/CE*

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***ANIOSPRAY QUICK** is labelled in accordance with the European regulation related to the classification and labelling of chemical products*

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Pure product ready to use :

- Storage between +5°C and +25°C.
- Stability :
 - Not opened flask : 3 years from the manufacturing date indicated on the label.
 - Opened flask: preserving of the 3 years if the product is stored in a container closed by the sprayer.

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Quantity of active substances per 100 g:

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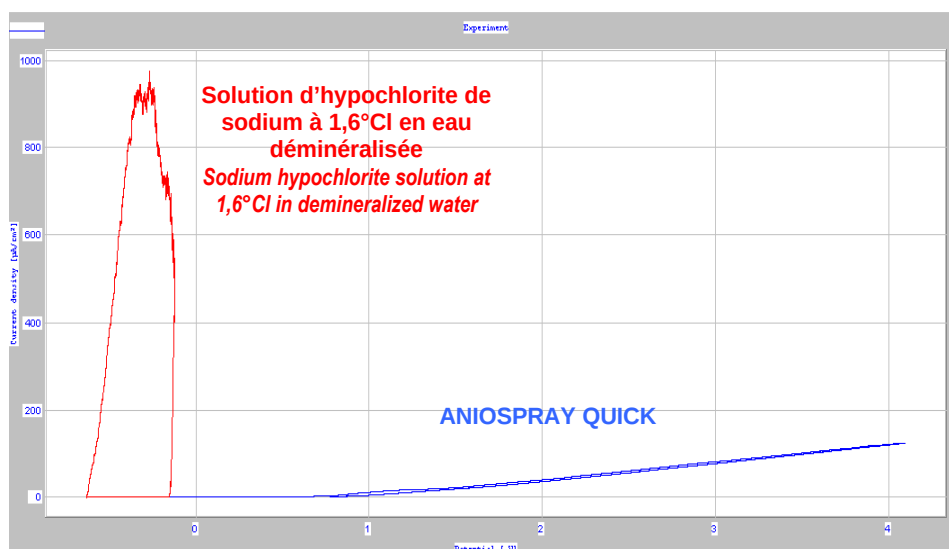
It is demonstrated, by electrochemical study of pitting corrosion according to the NF S 94-402-1 standard (May 2004), that ANIOSPRAY QUICK does not present pitting corrosive character with stainless steel Z30 Cr13 in the conditions of use recommended by Laboratoires ANIOS.

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Not concerned by biodegradability notion

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*Readily biodegradable substances and/or inherently biodegradable*³

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Substances which not fulfil the biodegradability criteria, or substances without any biodegradability data.

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ANIOSPRAY QUICK contains 100 % of inorganic material and of biodegradable organic material.

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Eye irritation, Category 2 (Eye Irrit. 2, H319).*

Environment

Hazardous to the aquatic environment – Chronic hazard, Category 3 (Aquatic Chronic 3, H412).*

* Classification drawn up in accordance with European regulation related to the classification and labelling of chemical products.

PERSONAL PROTECTION UNDER THE CONDITIONS OF THE INTENDED USE

Refers exclusively to this product. If the product is handled and/or exposed simultaneously with other chemical agents, these must be taken into consideration when choosing personal safety equipment.

Respiratory protection

In case of insufficient ventilation, with the risk of exceeding the TWA/STEL**, wear suitable breathing apparatus (type A2P2 filters).

Hand protection

When handling, wear suitable gloves (neoprene or nitrile)

Gloves must be replaced immediately if they show signs of wear and tear.

Eye and face protection

Avoid all contact with eyes.

Make safety goggles with side protection available to the personnel.

Water source nearby.

** please refer to SDS §8 for TWA (Time-Weighted Average)/STEL (Short-term exposure limit) of concerned substances present in the product

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The Safety Data Sheet (SDS) of this product is available on our Web site www.anios.com. In addition to visualization and/or impression, it is possible for you to download it.

The consultation of this SDS via internet enables you to receive later on by email any updated version of this document.

In the absence of an Internet access, the SDS remains available on request near our laboratories (tel.: +33 3 20 67 67 67 - fax: +33 3 20 67 67 68)

* an access is also possible at the site <http://www.quickfds.com>